
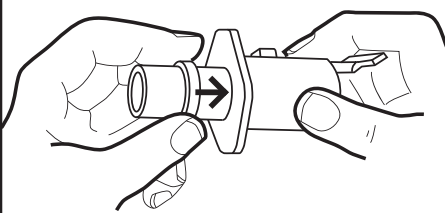
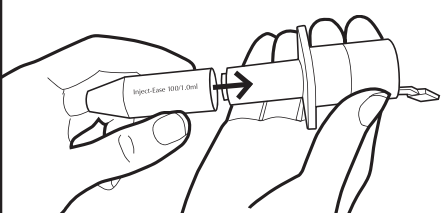
1


2

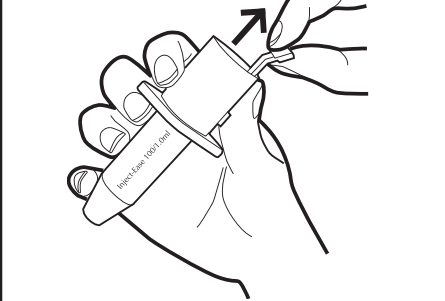
3


4

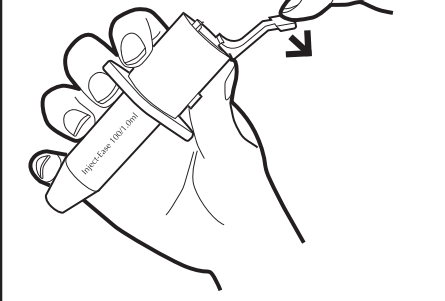
5

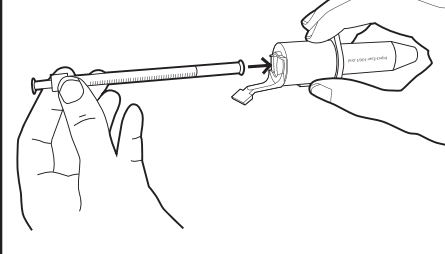
6

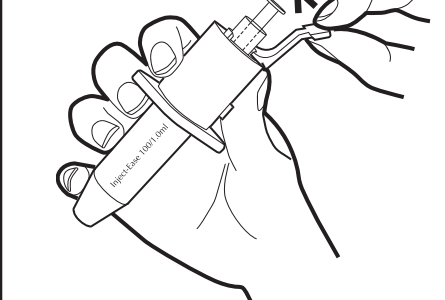
7

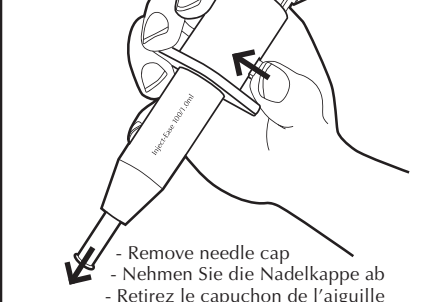
8

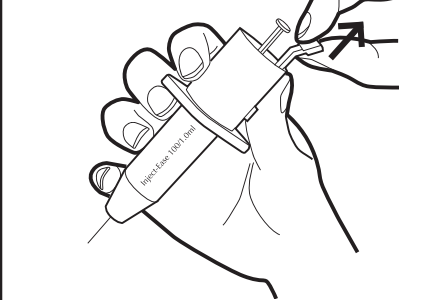
9

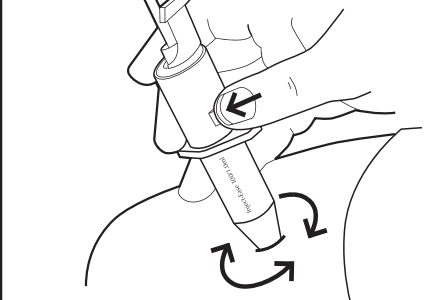10

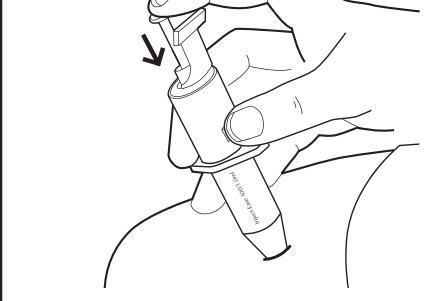11

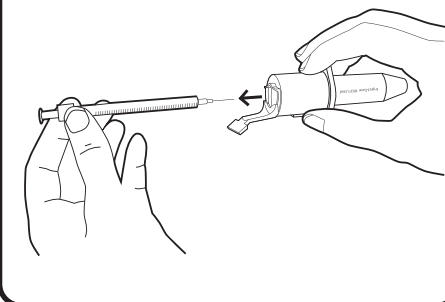Instructions for Inject-Ease®

The Inject-Ease® is a single-user device that assists with self-injections by automatically introducing the syringe needle into the skin. The Inject-Ease is for use with Becton-Dickinson (B-D®) 100 unit (1.0 ml, 1cc); 50 unit (0.5 ml, 1/2cc); or 30 unit (0.3 ml, 1/3cc) insulin syringes for subcutaneous injections and with the B-D® 100 unit (1.0 ml, 1cc), 5/8 in. tuberculin syringe (call Palco Labs Customer Support at 1-800-346-4488 or 1-831-476-3151 for the adapter tip that accommodates this syringe). The Inject-Ease is not intended to be used for intramuscular injections.

Cautions

- The Inject-Ease is intended for the single user. To avoid exposure to contaminants, do not use the Inject-Ease on multiple users.
- Consult your health care professional for guidelines on self-injection.
- Prior to use, read all directions and practice self-injections with a needleless syringe.
- For proper needle penetration, place the appropriate adapter tip on your Inject-Ease for the 100 unit (1.0 ml, 1cc); 50 unit (0.5 ml, 1/2cc); or 30 unit (0.3 ml, 1/3cc) insulin syringe. The adapter tip can be removed by pulling it off with a slight twist.

Instructions for Use:

Before you begin, wash your hands and the injection site and remove the plunger cap from the Inject-Ease. Follow the steps below. Each step corresponds to the illustration on the left.

1. **Decreasing Injection Depth (OPTIONAL):** Included in this package are two spacer rings that measure approximately 1/8" and 1/4" (3.17 mm and 6.34 mm) in width. Slide the appropriate ring onto the body of the Inject-Ease before attaching the adapter tip. This may be preferred by children or adults with thin skin or subcutaneous tissue. Consult your health care professional to determine the appropriate depth for you.
2. Place appropriate adapter tip on the Inject-Ease for the 100 unit (1.0 ml, 1cc); 50 unit (0.5 ml, 1/2cc); or 30 unit (0.3 ml, 1/3cc) insulin syringe.
3. Pull the latch (T-bar) upward to cock the device.
4. Push the latch (T-bar) backward to open the locking mechanism.
5. Insert the syringe containing your pre-measured dose with the **needle cap on** to prevent needle contamination.
6. Push the latch (T-bar) forward to lock the syringe in place.
7. Press the trigger button to expose the needle cap. Remove the cap and set it aside.
8. Pull the latch (T-bar) upward to re-cock the device.
9. **Hold the Inject-Ease so your thumb is on the trigger.** Place the device next to your skin at the injection site and rotate it a few times to desensitize the skin. Now press the trigger button.
10. Push in the plunger of the syringe with your index finger. Injection is now complete. Remove the device from your skin.
11. Pull the latch (T-bar) upwards to cock the device and then pull it backward to unlock the syringe. Lift the syringe from the device and discard in an appropriate sharps or biohazard container.

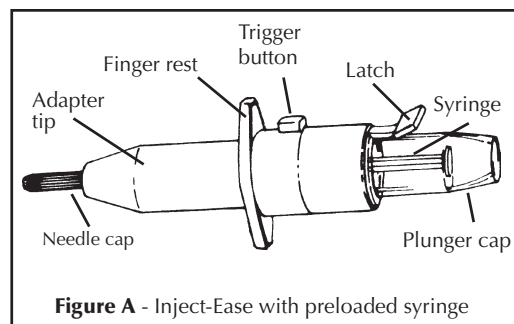
Figure A - Inject-Ease with preloaded syringe

Aspiration

If your health care professional advises you to aspirate prior to injecting, you will need clear tips instead of the opaque tips that come with this device. Please call Palco Labs Customer Support to obtain clear tips at no charge (30 unit clear tips are not available).

Pre-Loading

WARNING: Consult your health care professional to find out if storage of pre-loaded syringes is appropriate for the type of medication you use.

Certain medications can be preloaded in syringes and kept inside the Inject-Ease until needed. To pre-load, follow steps 1 through 7 in the *Instructions for Use*. Do not remove the needle cap until ready to use. Place the plunger cap over the plunger. To give the pre-measured dose just remove the plunger and needle caps and resume at step 8.

Cleaning

The Inject-Ease can be wiped off with a clean damp cloth or washed with warm soapy water and air dried. To avoid rusting the internal spring, do not submerge the device in water or wash the device in the dishwasher. Adapter tips can be submerged in water after removing them from the device.

Warranty

This Palco Labs product carries a five (5) year warranty against manufacturer's defects in materials and workmanship and against breakage or failure under normal use. This warranty does not cover misuse or abuse. Palco Labs will, as its option, repair or replace this product at no charge. This warranty gives you specific legal rights, and you may also have other rights under law.

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Inject-Ease is a registered trademark of Palco Labs, Inc. • U.S. Patent No. 4,787,891. Other International Patents Pending. All rights reserved. *Other trademarks are property of their respective holders
MDSS GmbH