

Inject-Ease®

Injections Made Easy

Conquer the anxiety of injections with the **Inject-Ease®**, the device that makes injections easy. Simply place your loaded syringe in the **Inject-Ease**, place the tip against your skin, and press the button to automatically deliver the needle through the skin. You control the rate at which the medicine is injected. The special tip is designed to help reduce the pain of injection. Making injections easier helps reduce anxiety - an independent clinical study showed that 91.6% of participants reported less fear of injections with the **Inject-Ease**¹.

From the people at Palco Labs, makers of world-class medical device products.



Key Features/Benefits

- Complete needle coverage - Reduces fear of injection by hiding the needle
- Special tip - Designed to help reduce pain through pressure anesthesia
- One-button ease of use - Provides independence to those with limited dexterity and coordination
- Designed for one-handed use - Allows access to harder to reach injection sites
- One-press needle insertion - Insert the needle through the skin automatically and easily
- Designed for BD* insulin syringes - works with 30, 50 or 100 unit syringes
- Comes with spacer rings for customization of injection depth
- Convenient for travel - maintains preloaded syringe integrity


Key Selling Points

- Industry-leading 5 year warranty
- Usable with subcutaneous self-injection treatments, including diabetes and other conditions²
- Palco Labs-recognized quality
- Affordable

¹ Clinical study available at www.palcolabs.com/injections/iestudy.html

² Consult your physician regarding use of the Inject-Ease with your condition.

Specifications

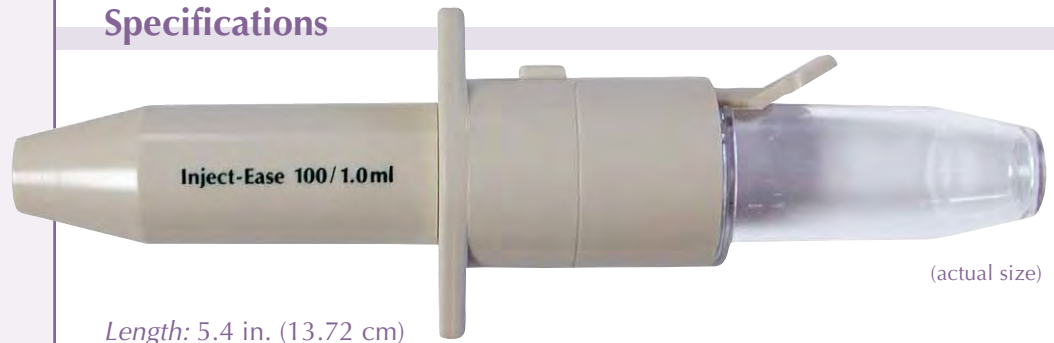

(actual size)

Ordering Information

Palco Order Hotline: 1-800-346-4488
Order Fax: 831-430-1610
E-mail: orders@palcolabs.com

Please note that not all products are available in all markets. For full details, please contact your local reseller or Palco Labs.

Special Orders

For special configurations and high volume orders, contact Palco Labs or send e-mail to info@palcolabs.com

Length: 5.4 in. (13.72 cm)

Width (at widest point): 1.7 in. (4.31 cm)

Materials: Acrylonitrile Butadiene Styrene (body and adapter tips), clear Polystyrene (plunger cap) – **Latex free**

Injection Type: Subcutaneous

Syringe Compatibility:

BD* 30, 50, or 100 unit insulin syringes and the BD* 100 unit 5/8 in. tuberculin syringe (must order adapter tip separately). Not for use with Monoject*, ReliOn*, AccuSure* syringes, or intramuscular syringes.

Package includes: 2 spacer rings – 1/8 in. (0.317 cm) and 1/4 in. (0.635 cm) – to decrease injection depth.

Cleaning:

Wipe off with damp cloth or wash adapter tips in warm soapy water; air dry. Do not autoclave, wash in dishwasher, or submerge unit in water.

Warranty:

Palco Labs warrants that each new Inject-Ease shall be free of all defects and guarantees normal performance for a period of 5 years.

Available Packaging Configurations



Inject-Ease is a registered trademark of Palco Labs, Inc. All rights reserved.

*Other trademarks are property of their respective holders.

PN/REF: IE400

Contents: Clamshell package with (1) Inject-Ease, 3 adapter tips (30, 50, & 100 unit), 1/8 in. (0.317 cm) and 1/4 in. (0.635 cm) spacer rings, clear plunger cap, tray filler card and instructions for use.

UPC Code: 0-25294-87400-7

PALCOLABS

360 El Pueblo Road, Suite 102
Scotts Valley, CA • 95066 USA
Ph: 1-831-430-1600 • Toll-free: 1-800-346-4488
e-mail: info@palcolabs.com
web: www.palcolabs.com

Specifications are subject to change without notice.
© 2006 Palco Labs, Inc. All rights reserved.

1001-35302-001